


# Quick Guide


How to use your 'Beato' smartphone glucometer

**STEP 1**

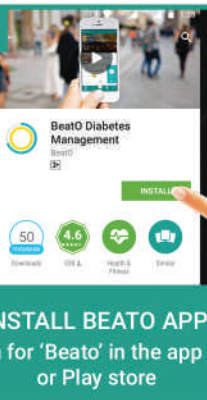

**INSTALL BEATO APP**  
Search for 'Beato' in the app store or Play store

**STEP 2**

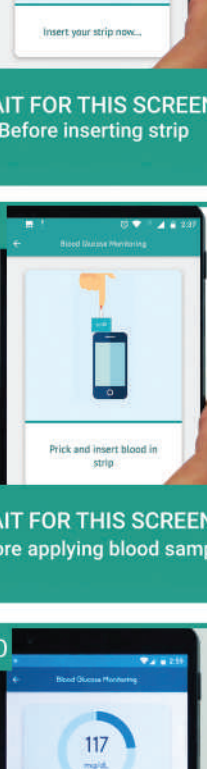


**LOGIN TO BEATO APP**  
Login to 'Beato' app and get started

**STEP 3**



**INSERT GLUCOMETER**  
Insert glucometer into headphone jack of your smartphone

**STEP 4**

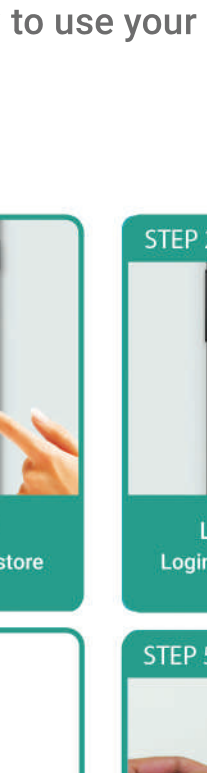

**WAIT FOR THIS SCREEN**  
Before inserting strip

**STEP 5**




**TAKE TEST STRIP OUT**  
Take the single-use test strip out of the foil package

**STEP 6**



**INSERT STRIP FRONT-FACING**  
Insert strip into the port of the docked glucometer

**STEP 7**

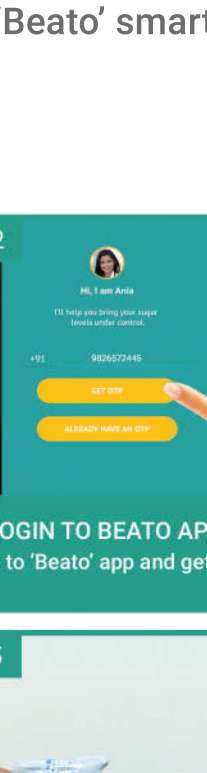

**WAIT FOR THIS SCREEN**  
Before applying blood sample

**STEP 8**




**PRESS TRIGGER**  
Place lancing device and press trigger for taking blood sample

**STEP 9**




**APPLY BLOOD**  
Apply blood on test strip for glucometer to analyze sample

**STEP 10**




**RESULTS**  
Your result will appear within 10 seconds

**STEP 11**




**CHANGE READING TYPE**  
Change reading type if needed by pressing drop-down menu

**STEP 12**



**SYNC**  
Sync your reading and get educator support

**STEP 13**



**ANALYSE**  
Click graph to view trends

+91 7863023286  
glucosupport@beatoapp.com  
www.beatoapp.com