


Quick Guide

How to use your Lancing Device

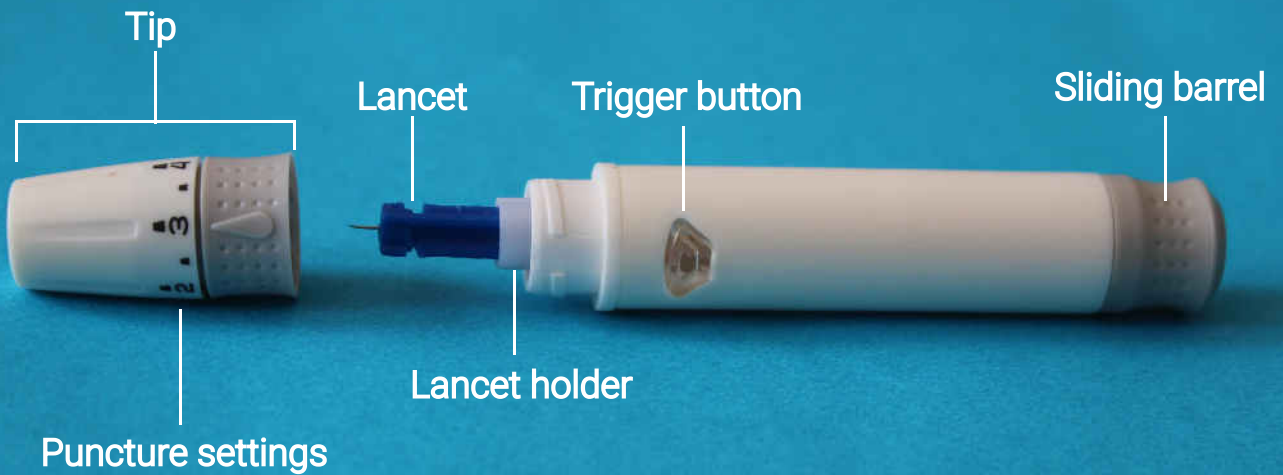

STEP 1



UNSCREW LANCING DEVICE

Unscrew the tip of the lancing device by turning it counter-clockwise

STEP 2



PLUG IN LANCET

Hold the sterile lancet and push it into the lancet holder

STEP 3



PULL PROTECTIVE DISC

Twist the disc off from the lancet

STEP 4



SCREW CAP BACK ON

Put the tip of the lancing device back on and turn it clockwise

STEP 5



PUNCTURE SETTINGS

Settings are from 1 (shallow) to 5 (deepest). Choose a number based on your skin type

STEP 6



ADJUST PUNCTURE SETTING

For first time usage, twist the cap and choose 3 (moderate) till you find the right setting

STEP 7



TRIGGER BUTTON INDICATOR

Ensure the trigger button has turned red. If not, read next step

STEP 8



TRIGGER BUTTON

If it has not turned red, pull sliding barrel and re-check

STEP 9



TAKE SAMPLE

Hold lancing device in position, press trigger and take sample